





## BIPOLAR FORCEPS

### Why spend \$6,000 on a bi-polar machine?

Plug the bipolar forceps into your Macan *Radio Surge* unit. Set the waveform to coagulation. You can now **achieve precise coagulation** in a **wet or bloody field** the same as with the very expensive bipolar units. The forceps and cord are autoclaveable.

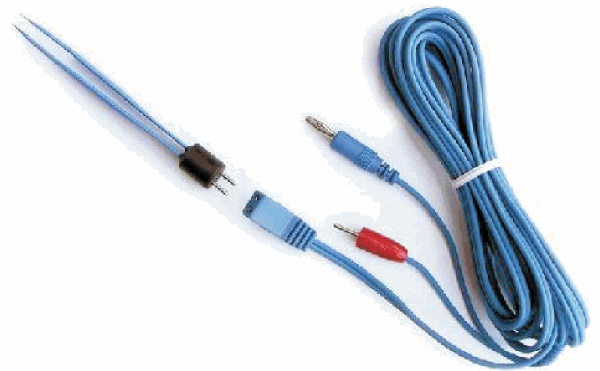

Remove the handpiece and dispersive plate from their respective ports. The bipolar forceps has one cord that splits into two end plugs. Connect the blue plug to the handpiece port and the red plug to the dispersive plate port. **In seconds, your Macan unit becomes a bipolar coagulator.**

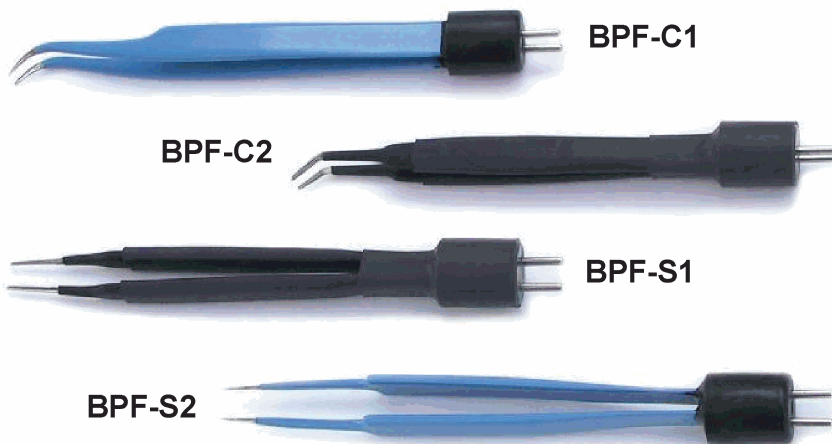

#### SPECIFY

**Curved or Straight Forceps**

**Serrated or Smooth Tip**

The smooth forceps taper to a fine point. The serrated forceps have blunt tips.

**Call Your Dental Dealer or Macan  
for Questions, Pricing & Ordering Information  
Po Box 166 Milton DE 19968  
Phone- 302.645.8068 Fax- 302.645-7049  
Email- [Info@macanmanufacturing.com](mailto:Info@macanmanufacturing.com)  
Website- [www.MacanManufacturing.com](http://www.MacanManufacturing.com)**